


EN Defibtech DDU-2000 Series: Selecting an Alternate Spoken Language

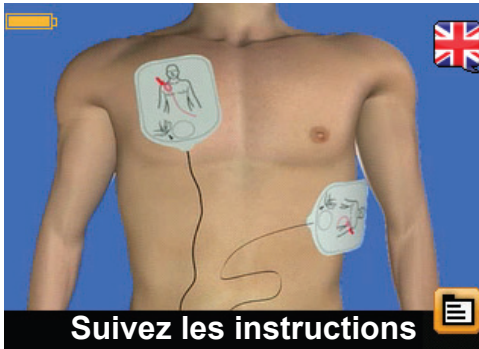


Some AED models are factory configured to support an alternate spoken language. If the AED supports an alternate language, a **Language Softkey Icon** (represented in the form of a flag) will be displayed. When the Language Softkey Icon is present, the user may press the corresponding softkey to switch the spoken voice prompts to the alternate language.

(Note: Text prompts continue in the primary language.)

Pressing the Language Softkey Icon again will switch the spoken language back to the primary language. The Language Softkey Icon is available only until the defibrillation pads are connected to the patient.

FR La gamme DDU-2000 de Defibtech: Sélection d'une autre langue pour les invites vocales

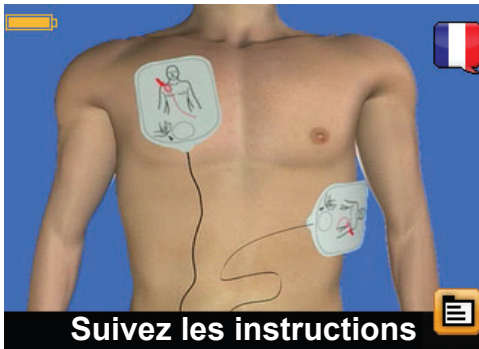

Certains modèles de DAE sont préconfigurés pour prendre en charge les invites vocales dans une autre langue. Si le DAE prend en charge une autre langue, une **icône de sélection de la langue** (sous forme de drapeau) est affichée. Si cette icône est affichée, l'utilisateur peut appuyer sur le bouton programmable correspondant pour passer aux invites vocales dans l'autre langue.

(Remarque : les invites textuelles sont toujours affichées dans la langue principale.) Appuyer de nouveau sur l'icône de sélection de la langue pour revenir à la langue principale. L'icône de sélection de la langue est disponible uniquement lorsque les électrodes de défibrillation sont connectées au patient.